

9121/9121-R

Shower Seat Assembly Instructions



Parts out of the box: (Be sure all parts are included before assembling)

1. Seat




2. Legs



(shown with adjustable legs attached)

3. Hardware

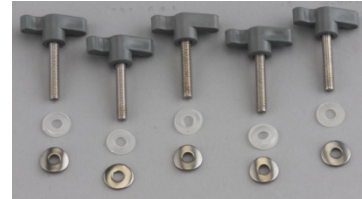

(Bench can be assembled in a variety of ways)

It is recommended to assemble bench near the bathtub where it will be used.

1



Place seat on a flat surface with the top facing down.

2



First place a plastic washer then a shaped metal washer onto one of the assemble screws. (The metal washer should contour to the shape of the tube.)

3



Place the first leg frame (with flattened middle section) into one of the molded guide slots.

4



Place the second leg frame over it and into the opposite molded guide slot.

5



Insert the assembly screw into the middle hole and tighten.

6



Repeat Step 2 for each of the remaining screws and insert a screw with washers into each of the remaining holes. (Be sure all 5 screws are secure and tightened.)

7. Turn the seat over and test for stability. (If not stable then check to make sure all the screws are and legs are adjusted to the same height.
8. Use the push buttons to adjust the seat to the desired height. Be sure the seat is level and stable before using and that all push buttons are fully engaged.



IMPORTANT

**READ THESE DIRECTIONS CAREFULLY BEFORE USE.
FAILURE TO DO SO MAY RESULT IN SERIOUS INJURY.
WEIGHT CAPACITY NOT TO EXCEED 300 LBS.**

Nova stands firm on our commitment to providing the highest quality products, as well as exceptional service. This product was built to precise standards and thoroughly inspected prior to shipment. This warranty represents our confidence in the materials and workmanship of our product.

Nova extends this warranty only to the original purchaser of this product. The warranty does not extend to any subsequent purchaser or owner. The warranty is void upon any sale or transfer of ownership or use by other person. Nova warrants its products are free from defects in material and workmanship under NORMAL use for the period commencing upon the date of purchase and continuing for the following specified period of time after that date:

**Metal Frame: Limited Lifetime Warranty
Non Metal/Metal parts: 1 Year Warranty
(Rubber tips, plastic or metal hardware)**

This warranty does not cover problems due to user negligence, misuse of the product, failure to adhere to product instructions, or normal wear and tear. Items that have been subject to negligent abuse, modified without prior written authorization are not covered within this warranty.

For warranty service, please contact the dealer from whom you purchased your Nova product. In the event that you do not receive satisfactory service, please call us directly at the number below. Be prepared to indicate the nature of the defect, the product serial number, and the name and location of where you purchased the product. You will also be required to provide a receipt of your purchase. **DO NOT** return any products to our office without prior consent. Repair or replacement is the only remedy under this limited warranty. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of product.

THIS WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, NOVA SHALL NOT BE LIABLE FOR ANY SEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.