

# 9026A

## Shower Bench with Arms & Back Assembly Instructions



### Parts out of the box:

#### 1. Leg posts and Back Tubes



#### 2. Seat & Back




#### 3. Legs




#### 4. Legs Guides




#### 5. Screws

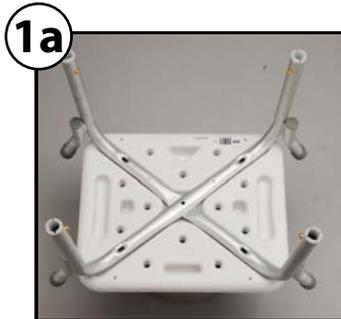


Set the seat on an elevated surface face down and place the leg marked 2 into the diagonal inset on the bottom of the seat. (The #2 on the seat should line up with the #2 label on the tube. Repeat for tube labeled #1 so that tube #2 is under tube #1. Insert #1 screw w/ washer into the center hole and tighten.

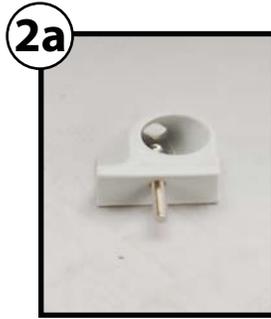

Insert four #2 screws (with smaller head) into the plastic tube guides.

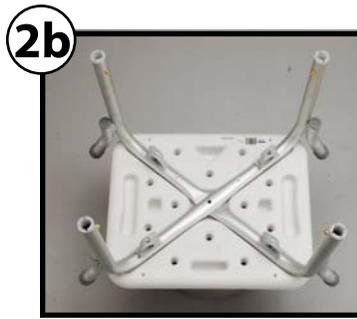


Place the tube guides onto the legs so that the screw goes through the tube and into the seat bottom.

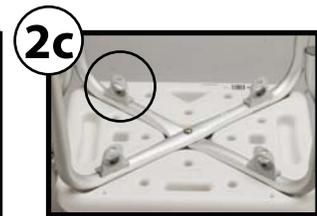


Be sure the guides are facing in the correct position so that the back tubes can be inserted through them in step 5. Tighten all screws securely once the guides are determined to be in the correct position.



Attach the legs by telescoping the adjustment leg over the leg stem and pushing the push pin in. Be sure to adjust all four legs to the same adjustment holes.

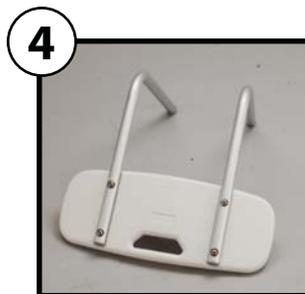

Attach the back tubes to the backrest using four #1 screws with washers.

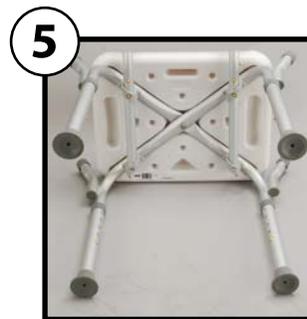

Lay the bench on the floor so that the back of it is facing upward. Place the back tubes through the rear tube guides. Depress the brass push pins and insert the end of the tubes through the front tube guides until the pins engage into the rear tube guide. (The back can be adjusted to a deep or shallow position depending on user preference.) Be sure pins are fully engaged by pulling on the back.

# 9026A

## Shower Bench with Arms & Back

### Assembly Instructions *(cont.)*



6



Set the bench on the floor and insert the arms by depressing the brass push button and inserting each side into the arm stems. Be sure pins are fully engaged by pulling the arm.

7



Use the push buttons on the legs to adjust the bench to the desired seat height.

**BE SURE THE BENCH IS LEVEL AND STABLE BEFORE USING.**



#### **IMPORTANT**

**READ THESE DIRECTIONS CAREFULLY BEFORE USE.  
FAILURE TO DO SO MAY RESULT IN SERIOUS INJURY.**

**WEIGHT CAPACITY NOT TO EXCEED 275LB.**

#### **LIFETIME LIMITED WARRANTY**

Nova Ortho-Med, Inc. warrants this product to be free from defects in material or workmanship for a period of lifetime from date of purchase. If within the specified warranty period this product is proven to be defective, it will be repaired or replaced at NOVA O.M.'s option. This warranty does not cover shipping costs incurred in replacement or repair of this product.

NOVA O.M. stands firm on our commitment to providing the highest quality products, as well as exceptional service. This product was built to precise standards and thoroughly inspected prior to shipment. This Lifetime Limited Warranty represents our confidence in the materials and workmanship of our products. Do not return any products to our office without prior consent.

This warranty does not cover problems due to user negligence or misuse of the product. This warranty shall not extend to non-durable components, including but not limited to rubber tips, hand grips, casters, and other parts subject to normal "wear and tear."

THIS FOREGOING WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. NOVA O.M. SHALL NOT BE LIABLE FOR ANY SEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.