

UpLift Seat Assist

Product SKU Number:

MEDUL100 (Supports users weighing 80-230 lb)

MEDUL300 (Supports users 195-340 lb)

Description: Uplift Seat Assist is a mechanical lifting seat that you can take with you wherever you go. It will ease you into your seat, as well as help you up. The Seat Assist automatically supports you as you begin to stand, and can lift 70% of your weight. Because it is non-electric, it can be used safely inside or outdoors. It is best for use on firmer seating surfaces, such as regular chairs, park benches, and theater seats. In the market place for mobility equipment, Uplift Seat Assist is unique, offering ultimate convenience and total portability. If you currently own a seat lift chair, Seat Assist can serve as a valuable aid to your independence in other rooms of your home, or when you go out and about. 19" L x 17" W x 4" H. (9 lbs.).

Product Features:

1. **Safe:** The flexible seat curves to a flat surface that lifts you up on a level plane. This patented LevelLiftT design provides for a safe lift up, with no forward dumping motion. Patent # 5,316,370
2. **Durable:** The injection-molded base is made of durable polycarbonate, and covered with a non-slip material that prevents shifting on fabric seating surfaces.
3. **Waterproof:** The breathable fabric cover is waterproof and machine washable in cold water. Comes in black.
4. **Easy Care:** The foam cushion has a water resistant coating that can be wiped down with mild soap and water.
5. **Memory Foam:** Slow-release memory foam helps protect against pressure sores and conforms to the shape of your body.
6. **Self-Powered:** A hydro-pneumatic gas spring serves as the lifting mechanism, releasing automatically when you begin to stand. No battery or electricity required.
7. **Adjustable Lift:** Weight setting grooves provide easy adjustment for users of varying weights. Choose Model MED-UL 100 if you weigh 80-230lb (36-159kg) or Model MED-UL 300 if you weigh 195-350lb (88-159kg).
8. **Portable:** When the lifting mechanism is disengaged, the cushion folds flat for easy transport. The built-in handle on the base makes it easy to carry Uplift Seat Assist anywhere.
9. **Easy to Use:** To get up you simple lean forward to stand and the lifting mechanism automatically releases to provide assistance - lifting as needed, up to 70% of your body weight.

Pictures of Product:



Product Classification: Mobility Aids, Lifting Seats