Measuring your stairs for an AmeriGlide stair lift is not a difficult job. You will need a good metal tape measure and a pen or pencil to record your measurements. Please print this page for use as a reference throughout the measuring process.

The most important step for stair lift installation is to decide which side of your stairwell you want the stair lift on. To avoid confusion, please stand at the bottom of your stairs and look up at the top of the stairs. Which side will the stair lift go on, the left or the right? AmeriGlide stair lifts can be installed on the same side as the banister and can even be installed where there is no wall because the stair lift attaches to the floor.

Please note that the stair lift rail will stop about 14 inches beyond the bottom step and extend 6 inches past the top step. Take caution that this will not interfere with any doorways at the top or bottom of the stairs.

After deciding which side you will mount your stair lift on (looking from the bottom to the top), please follow these instructions for accurate measurements. We recommend that you follow the advice to always "measure twice". Please note that all measurements should be recorded in inches.

MEASUREMENT 1: This is the most critical measurement. Please stand at the top of stairs and extend your tape measure down to the landing, letting it rest on edge of each tread along the way. The tape should not sag or bend along the way. Please record the distance from the edge of the upper landing to where the end of the tape measure rests on the landing at the bottom of the stairs. Please refer to the diagram for assistance.

MEASUREMENT 2: Please measure from the edge of the upper landing (like Measurement 1) to the edge of the first stair's tread at the bottom. Please refer to the diagram for assistance.

MEASUREMENT 3: Please measure the width of your stair's treads. Please refer to the diagram for assistance.

MEASUREMENT 4: Please measure from the edge of the first stair at the bottom to the wall (or door), if there is one. Please refer to the diagram for assistance.

MEASUREMENTS 5-7: These are the tread measurements. Please refer to the diagram for assistance.

MEASUREMENTS 8: Measure from the top step to the nearest door, wall or other obstruction at the top of the stairs.

Measurements	
Which side (looking from the bot- tom to the top)?	
#1 Overall Length	inches
#2 Top Stair to Bottom Stair	inches
#3 Staircase Width	inches
#4 Bottom Clearance	inches
#5 Rise of One Step	inches
#6 Tread of One Step	inches
#7 Diagonal of One Step	inches
#8 Top Clearance	inches









